

CubitiROM - Elbow brace

ART.NO 20202

CubitiROM is used for post-operative or post-traumatic stabilisation of arm and elbow to provide a controlled range of movement or immobilisation. The stop point for flexion can be set at 0–120° and the stop point for extension can be set at 0–90° – at 10° intervals in both directions. The length of the upper and lower arm splint can be adjusted using the telescopic mechanism of the brace. The length of the brace can be adjusted between 36 and 45 cm. Use the shoulder strap if extra relief is required. CubitiROM is comfortable to wear and sits snugly against the arm without migrating. Light, airy and easy to put on and adjust.

Indications:

Used for post-operative or post-traumatic stabilisation. Muscle/ligament injuries. Distal fracture of the humerus. Elbow plastic. Stable elbow fracture.

Storlek:

Art. no.	Size	Side
20202-16-50	Universal	Left
20202-26-50	Universal	Right

Material:

Aluminium splints, polyester/cotton padding, Velcro.

Washing instructions:

Wipe down splints/joints with soap and water, or clean with spirits. Machine wash the textile parts at 60°.

Accessories:

Extra padding, item no. 20204 – order separately.



The length of the splints and the scope of movement of the joint are adjustable and can be adapted to suit each individual patient.

Length of the brace from the centre of the hinge.

Forearm	19 - 24 cm
Upperarm	18 - 21 cm

