

CubitiComfort - Humeral fracture brace

ART.NO 20222

Stable and comfortable full arm orthosis designed to be used in rehabilitation and/or treatment of fractures according to the Samiento principle. CubitiComfort eases pain through fixation, while still allowing mobility of the shoulder and elbow, which reduces the risk of stiffness and muscular atrophy during treatment. The mobility of the elbow joint can be adjusted through extension/inflexion in 15° intervals. The product can easily be customized as it can be fixated in the desired angle. The orthosis is comfortable to wear, the wide straps together with mold casted plastic tiles provides an effective and even compression around the upper and lower arm. The textile shoulder part enables a comfortable fit, while ensuring that the product stays in the correct position. The integrated plastic tiles may easily be taken out and adjusted/cut to the desired fit for the patient. The air gap in the fabric contributes to an even distribution of the compression around the upper arm. The product is constructed to fit both the left and right arm.

Indications: Diaphyseal humeral shaft fracture. Fracture close to elbow. Elbow ligament injury.

Contraindications: Massive soft tissue damage, poor compliance alternative. Unacceptable fracture position.


Material: Ventilating distance fabric which feels cool against the skin and allows the skin to "breathe". Polyester 59%, Polyamide 37%, Cotton 2%, Elastane 1%, Polyurethane 1%. Latex-free. Metal stay in aluminum.

Washing Instructions: Retire el soporte de metal mientras lava. Se recomienda el lavado a 40°C (104°F). Quite las placas y fije los cierres de velcro. Use una bolsa para lavar la ortesis. No use lejía. No la lave en seco. No la planche. No use una secadora. El soporte de metal puede limpiarse con un paño húmedo.

Measuring: Measure circumference around the upper and lower arm. The total length of the orthosis can be adjusted to the desired fit.

Additional tiles, art.no. 20221:

To be ordered separately. 4 pieces/package, size S – XL. Additional tiles may be used to attain stronger compression and stability or for treatment of patients with wider circumference of the upper arm. 1, or alternatively, 2 tiles to be placed in each of the side pockets.

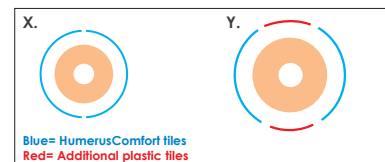

If necessary, extra tiles are applied to maintain circular compression for hefty arms.

	A. Circum. upper arm (cm)	B. Circum. lower arm (cm)	C. Orthos length upper arm	D. Orthos length lower arm	E. Orthos total length	Art.no
S	14-23	18-26	26,5	17	46-54	20222-09-40
M	23-32	23-32	28,5	19	50-58	20222-09-50
L	30-39	28-38	30,5	20	52-60	20222-09-60
XL	38-50	28-38	30,5	20	52-60	20222-09-70

MD Class I CE