

Vilmer - Compression vest

ART.NO 51103

Vilmer compression vest is used after breastreduction at gynecomastia to reduce swollenness and risk of bleeding. Adjustable wide shoulder straps, targeted compression zones, double material and firm compression over shoulders and chest for optimal compression. Zipper. Full length. Available in white: 51103-09 and black: 51103-00.

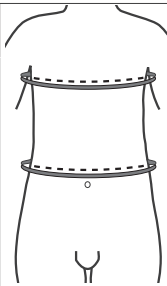
Indications: Vilmer is recommended after breast reduction at gynecomastia, with or without removal of the mammary gland. Worn after plastic surgery to ensure best possible result. Compression significantly reduces the risk of swollenness and bleeding after surgery, as well as enables a fast healing process and provides pain relief.

Materials: Polyamide 80%, Elastane 20%. Latex free.

Washing instructions: Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). The product will last longer if washed at a lower temperature. Dry flat. Do not tumble dry. Close the zipper before washing. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

Measurement: Fitting of the correct size for the patient is recommended to be made before surgery. The fit of the garment should be tight but comfortable. A few weeks after surgery, a smaller size may be used to attain desirable compression.

Keep in mind that compression garments are designed for a snug fit. Optimal compression and support require accurate measurements.

Size	Bust cm	Waist cm	Art.no white	Art.no black	
XS	81-86	73-78	51103-09-30	51103-00-30	
S	87-93	79-85	51103-09-40	51103-00-40	
M	94-100	84-90	51103-09-50	51103-00-50	
L	101-108	91-97	51103-09-60	51103-00-60	
XL	109-116	98-105	51103-09-70	51103-00-70	
2XL	117-124	106-113	51103-09-80	51103-00-80	



51103-09



Adjustable shoulder straps. Zipper with hook-and-eye and soft cover.



51103-00

Vilmer bak

