ART.NO 51203

Compression body with high back in knitted seamless material. Recommended after surgery or liposuction on the abdomen, sides and/or back. 3-step hook and eye closures in both sides for easy application and adjustable fit. Open crotch to facilitate use directly after surgery. Fitted legs with wide wedge on inside seam for comfortable wear. Shape knitted material in 4-way stretch follows the body contours. The stable, soft material provides evenly distributed compression without discomfort or digging in to the skin. Shape knitted zones over buttocks and groins for better fit. Reinforced knitting around abdomen for extra support.

Indications:Compression garment after plastic surgery. Corresponding class 1.

Contraindications: Infection. Arterial circulatory disorders. High risk of thrombosis.

Material: 87% Polyamide, 13% Elastane. Latex free.

Washing instructions:

Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). The product will last longer if washed at a lower temperature. Use a laundry bag. Do not use bleach or fabric softener as this can impair the function of the material. Do not tumble dry.


Measurement: For best fit and correct compression it is important that measurements are taken carefully and size chosen according to the measuring table. The fit of the garment should be tight but comfortable. **Waist:** Measure around the thinnest part of the waist. **Hips:** Stand with both feet together and measure the circumference over pubis, where hip/seat is at its widest. **Thigh:** Circumference measurements at the middle of the thiah.

Size	Waist cm	Hip cm	Thigh cm	Art.no Black	Art.no Beige
XS	62-68	86-91	46-50	51203-00-30	51203-08-30
S	69-75	92-97	50-54	51203-00-40	51203-08-40
м	76-83	98-103	54-58	51203-00-50	51203-08-50
L	84-92	104-111	58-62	51203-00-60	51203-08-60
XL	93-101	112-119	62-66	51203-00-70	51203-08-70
2XL	102-110	120-127	66-70	51203-00-80	51203-08-80



Hook and eye closure in both sides and a wide back.



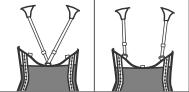

The shoulder straps can be attached at the front or in the sides.





MD Class I

CE

Compression class I

20240205_EN

NordiCare Ortopedi+Rehab AB Solrosvägen 1 • S-263 62 Viken, Sweden • +46 (0)42 35 22 20 • info@nordicare.se • www.nordicare.se