PlusPoint - Knee Brace

ART.NO 70653

The PlusPoint Knee Brace is suitable for those with moderate to severe ACL, PCL, MCL and LCL instabilities. Protection and restriction of ROM after ligament reconstruction, meniscus reconstruction or meniscus resection. The Knee Brace prevents hyperextension by limiting range of motion through flexion/ extension stops.

In addition, the Knee Brace has an anatomical design with a stable and light weight frame, easily shaped by a technician. Strap guides with anti-rotation system facilitate putting the brace on and adapt to the leg. Padding of straps are seam-free and without adhesive points for pleasant wearing next to the skin. Very flat polycentric joint with plug-in flexion/extension stops. A medial, anatomically formed condyle pad is pre-mounted, for the greatest in wearing comfort and optimum hold. Two sets of replaceable condyle pads of different thickness enable simple adaption. Internal frame pad secured with flat, welded press-studs for secure positioning. Anti-slip and skin-friendly cover made from multi-grip fabric.

PlusPoint can be used for bathing / rehab in fresh water. Make sure all parts are dry before using the orthosis again.

ROM control limitation:

Flexion Stops: 45-90° (in 15° increments) Extension Stops: 0-40° (in 10° increments) Immobilization: 0-30° (in 10° increments)

Indications: Moderate to severe ACL, PCL, MCL and LCL instabilities. Complex instabilities of the knee joint, traumatic or degenerative. For immobilization of the knee joint post-op/post trauma. Protection of the ligaments when doing sports.

Material: Aluminium splints. Polyester padding. Latex free.

Washing instructions: Wipe the hinge with a soft, damp cloth.

Size and measuring:

Size: S - 2XL Height: 40 cm Thigh circumference: Measure 15 cm/6 in above knee, center/ mid-patella.

| Left | | Right | | Thigh Circumference |
|------|-------------|-------|-------------|------------------------|
| Size | Art.no. | Size | Art.no. | in cm |
| S | 70653-10-40 | S | 70653-20-40 | 39-47 |
| Μ | 70653-10-50 | Μ | 70653-20-50 | 47-53 |
| L | 70653-10-60 | L | 70653-20-60 | 53-60 |
| XL | 70653-10-70 | XL | 70653-20-70 | 60-67 |
| 2XL | 70653-10-80 | 2XL | 70653-20-80 | 67-75 |





Anatomical design, stable and light weight frame, easily shaped and with very flat polycentric joint.





NordiCare Ortopedi+Rehab AB Solrosvägen 1 • S-263 62 Viken, Sweden • +46 (0)42 35 22 20 • info@nordicare.se • www.nordicare.se