

# Stadig - Ankle orthosis

ART.NO 80120

Stadig is a stable ankle orthosis to be used in case of acute injury or prolonged instability in the ankle joint. The orthosis can also be used for prevention. The soft padding conforms to the ankle's contour and provides good comfort and support. The slats are universal and fit both the right and left foot. Stable velcro strap fixes the orthosis in a correct position. For optimal function, the orthosis should be used in conjunction with laces, for example a walking shoes/gym shoes.

**Size:** Universal size.

**Indications:** Severe and moderate acute ankle sprains (functional and post-operative treatment), stable malleolar fracture, after care for fractures in the rehabilitative phase

**Contraindications:** None known.

**Material:** Nylon, Polyester, PU Foam, Fibreglass.

**Washing Instructions:** Remove the foam padding and the closure straps from the splint. Close the hook of the straps. Hand-wash the straps and foam in lukewarm water (max 35°C) using a mild soap. Rinse with water and allow it to dry at room temperature. Clean the plastic shells with a damp cloth. Be sure that the materials are thoroughly dry before re-applying.



MD Class I CE

Size	Color	Art.no
Universal	Black	80120-00-50