# INFORMATION FOR 20116 AbduSling - SHOULDER ABDUCTION 15°

# AbduSling

Art.no 20116

## Information

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Serious incidents shall be reported to NordiCare and the competent authority.

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## Composition

Textile materials except pillow: 2% polyamide, 26% polyurethane, 2% cotton, 70% polyester.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se).

NordiCare Ortopedi och Rehab AB  
Solrosvägen 1, SE-263 62 Väken, Sweden.  
Tel +46 42 35 22 20, Fax +46 42 35 22 21  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)

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


**EN** 1. Position the arm in **A** so that the elbow rests against the rear edge. Close with **C** on top.

2. Pull **D** through the slide buckle on **C**. Secure **D** approx. 15 cm above.

3. Pull **D** through the slide buckle on **A**. Open the crocodile clip. Cut **D** to the desired length. Refasten the crocodile clip to the strap and secure in place.

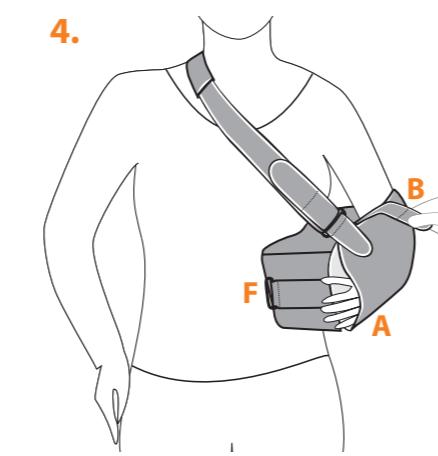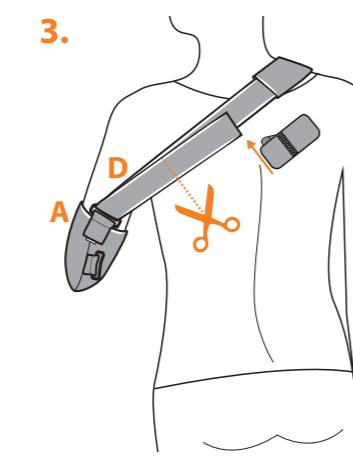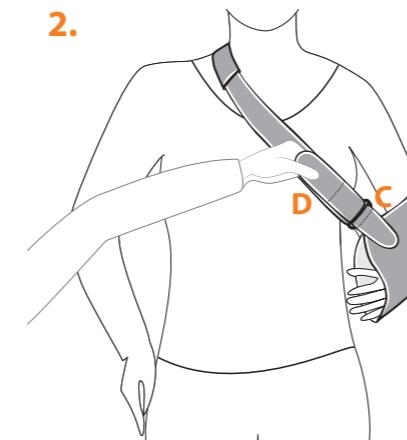4. Position **F** against the hip at an appropriate height, with the broader part facing forward. Secure **A** to the Velcro fastening on the pillow. Secure **B** on top of **A** to fix the arm in the position required.

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9. Tilpasset produkt forfra.

10. Tilpasset produkt bakfra.

**EN** 5. Draw **E** through the front slide buckle on the pillow. Secure **E** approx. 15-20 cm along the strap.

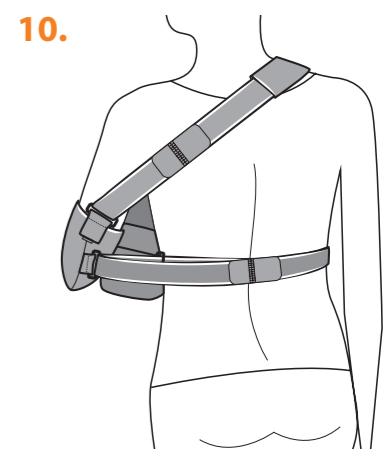
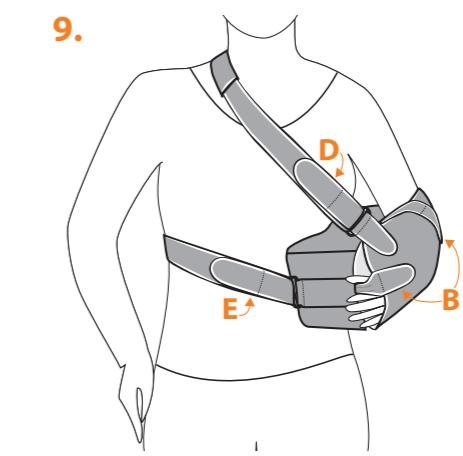
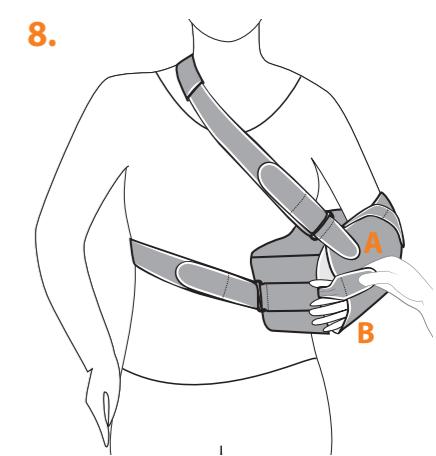
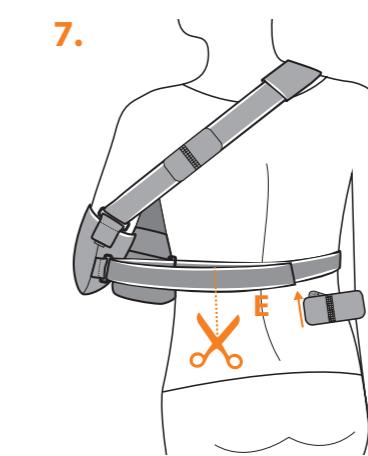
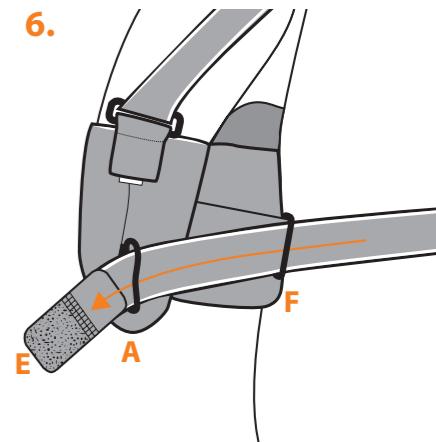
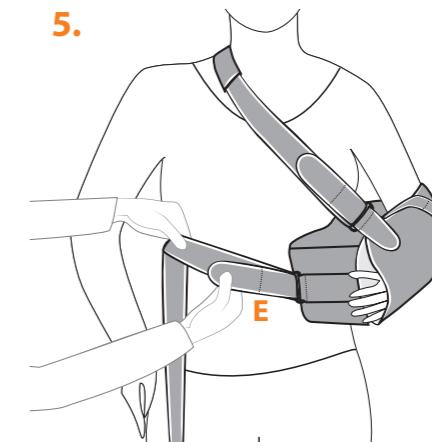
6. Draw **E** through the rear slide buckles on the pillow, first through the buckle on **F** and then through the buckle on **A**.

7. Open the crocodile clip. Cut **E** to the desired length. Refasten the crocodile clip to the strap and secure in place.

8. Secure **B** forwards over **A** as support for the thumb, if required.

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**EN** Open fastenings **B+B**, loosen shoulder strap **D** and waist strap **E** at the front.

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