INSTRUCTION FOR USE

DuoTrain – SHOULDER IMMOBILIZER

part no 20119

Package included

- A. Slinga och axelband / Slynge og skulder stropper / Fatle og skulderstropp / Kolmioliina ja olkahihna / Sling and shoulder strap
- B. Velcro-platta x 2 / Velcroplade x 2 / Velioplatte x 2 / Tarranauhoja x 2 / Velcro plate x 2
- C. Fäste / Lukkemekanisme / Feste / Kiinnikeet / Fastening
- D. Fäste med D-ring / Lukkemekanisme med D-ring / Feste med D-ring / Kiinnike ja lenkki / Fastening with slide buckle
- E. Midjeband / Taljeband / Midjebånd / Vyötäröhihna / Waist strap

Application

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EN Important before use:

- Gray velcro plates should always be used.
- The sling is mounted to fit the right arm, for use on left arm: turn/move waistband **E** and velcro plates **B**.
- 1. Place the arm in the sling so that the elbow rests against the rear edge. Close the top edge using **C + D**.
- 2. Thread the shoulder strap **A** through the D-ring at **D**. Make sure that the fastening **D** is located right above the wrist (so that the hand is supported).
- 3. Fasten waist strap **E** on the velcro plate **B** at the wrist, if necessary, move **B** to the desired location.
- 4. Adjust the length of the waits strap at the back.

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EN For relieve/immobilization/fixation of shoulder/arm. In case of trauma, emergency or postoperative care.

Contraindications: For patients with sinus carotid sensitivity avoid pressure from the orthosis in the carotid sinus area.

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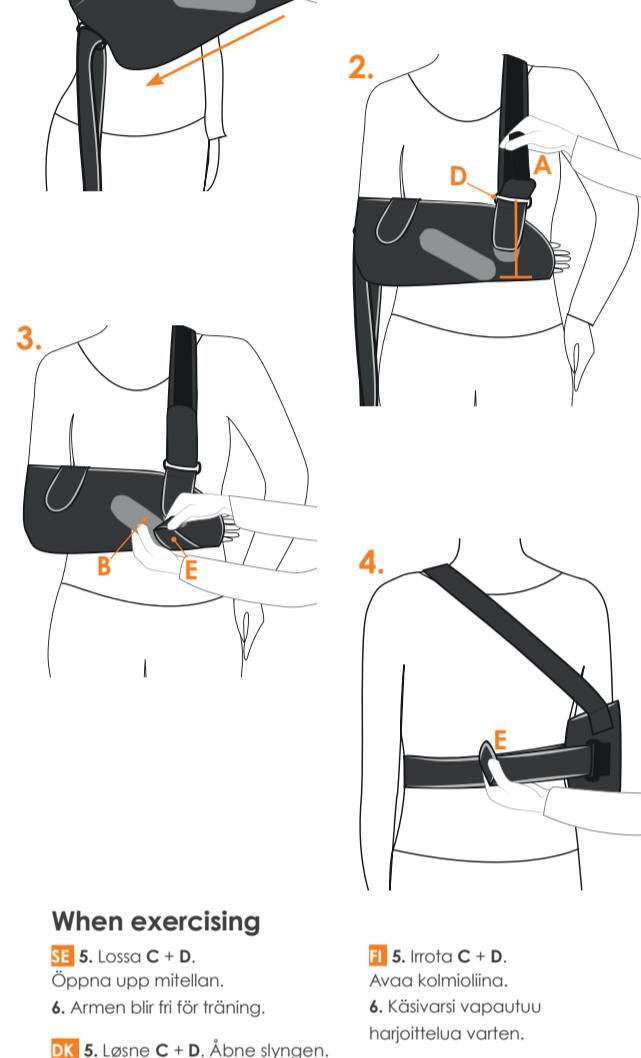
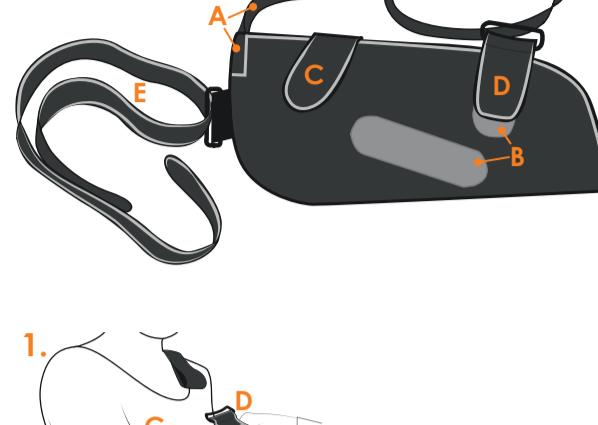
EN Periodically check the condition of the Velcro to ensure they are free from grease and damage. Remove DuoTrain before taking a bath/shower. Seal the Velcros before washing, machine wash at 40°C (104°F), can be washed at 60°C (140°F) if necessary. Use of a laundry bag is recommended.

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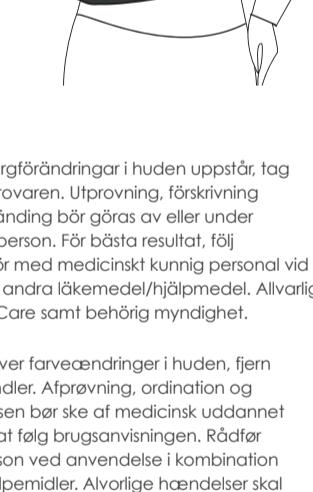
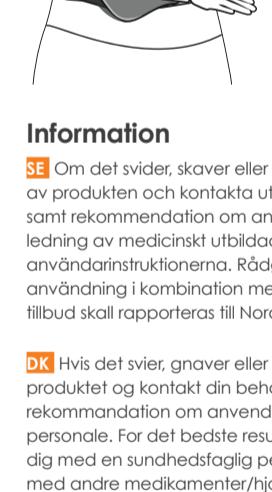
EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

Material:

Nylon 42%, PU-foam 23%, Polyester 21%, Cotton 14%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.