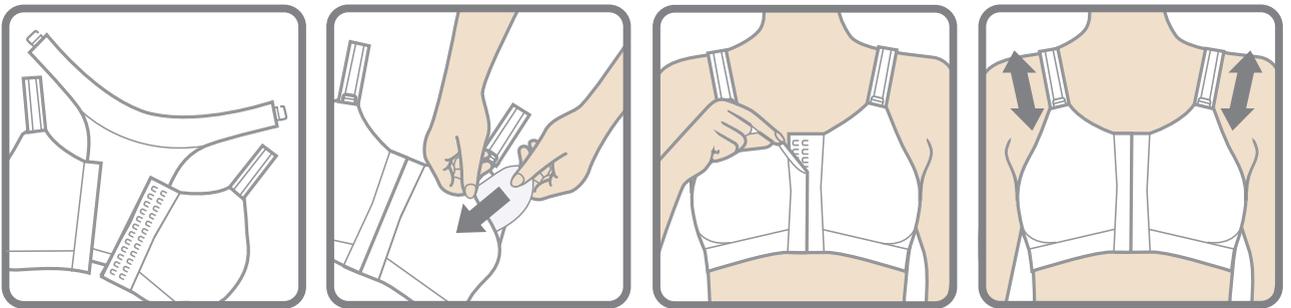
## INSTRUCTIONS FOR USE

## MIRABELLE, MIRABELLE QUEEN – POST SURGICAL BRA

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**EN** The bra can be opened both in front and shoulder straps, allowing easy application and adjustment after surgery and/or during recovery. The Pockets are to be used for placing padding/prosthesis.

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**EN** If you experience pinching, chafing, or color changes in the skin, remove the bra and contact the product fitter. If you recently went through breast surgery, fitting, prescribing and recommendations of use, should be done by or under the supervision of a medical professional. If you have any questions regarding use and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are placed correctly, to avoid pressure points and/or obstructed flow from the drainage tubes. For breast augmentation a stabilizer band for breast implants may also be required. The bra must be removed during radiotherapy as it contains metal parts. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Post-surgical bra with a measured compression level equivalent to compression class 1. Intended for use after different types of breast surgery. **Contraindications:** Infection. Arterial circulatory disorders. High risk of thrombosis.

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**EN** Use the size guide and size schedule to decide the right size. The fit should be tight but comfortable. The degree of compression can be adjusted by choosing a larger/smaller size.

**NOTE:** Due to possible post surgical swelling and edema, you may need to go up 1-2 sizes compared to your normal bra size. **Important:** Always measure yourself to find the right size!

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## Material composition

S-2XL: 82% Polyamide, 9% Polyester, 9% Elastane, Latex free.  
3XL-4XL: 88% Polyamide, 6% Polyester, 6% Elastane, Latex free.

## Washing instructions

**EN** Machine wash at maximum 60°C (140°F), for longer durability, 40°C (104°F) is recommended. Fasten hook- and- eye closure before washing. Use of a laundry bag is recommended. Possible to tumble dry at low heat. Do not iron. If the garment has any blood stains, soak or wash in cold water. Do not use bleach, chlorine or fabric softener as this could cause the material to lose functionality.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se).