## INSTRUCTIONS FOR USE 50204, 50205, 50209

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**EN 1.** Choose the correct size and put the binder in place. Close the Velcro at centre front.  
**2.** Adjust until comfortable. Make sure it is not too tight.

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**EN** Circumference in centimeters around the abdomen of intended placement.

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**EN** To be used in post-surgical treatment, after thorax or abdominal surgery in order to prevent pressure on wounds, relieve pain and in the early stages of mobilization of the patient. Also for different types of less severe hernia or to create a stoma binder.

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### Materials

Form 15: Polyester 65%, Polyamide 20%, Rubber 15%.  
Form 25: Polyester 76%, Rubber 16%, Polyamide 8%.  
Form 30: Polyester 78%, Rubber 17%, Polyamide 5%.

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**EN** The initial fitting, prescribing and recommendations for use should be done by/ under the direction of qualified medical practitioner or other professionals. Follow the instructions for best result.

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**EN** Periodically check the condition of the Velcro to ensure they are free from grease and damage. A qualified medical practitioner should supervise use in combination with other medical products.

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**EN** Machine wash at 40°C (104°F) (higher temperature is possible, but reduce the life time of the Velcro). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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