NordiCare®

Frej

art no 50253

HERNIA BRIEFS

2021-11-08

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EN To be worn instead of regular briefs. Flat pads (1) are included, convex pads (2) for increased pressure can be ordered separately. The trousers provides lift and support (3).

- Place the pads in the pockets. Convex pads should be positioned with the convex part against the body. Remove pads before washing briefs.
- Put on the briefs first thing in the morning. Preferably laying down, when the hernia is most sunken.
- To be worn directly against the skin. Do not use the briefs directly on damaged/irritated skin.
- Do not use with lotions or creams.

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EN For conservative treatment of light/medium inguinal hernia also postoperatively to support the area of surgery and provide pain relief. **Contraindication:** Incarcerated hernia.

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EN Prescription, initial fitting and treatment recommendations are to be made by a qualified medical practitioner. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for use. Serious incidents shall be reported to NordiCare and the competent authority.

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
EN Hip circumference (B) in cm. Height is measured from pubis and up (A).

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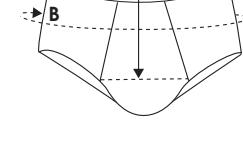EN Machine wash at 40°C (104°F). Use washing bag. Do not bleach. Do not dry clean. Do not tumble dry.

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Materials

Cotton 88%, Elastan 12%. Pads in Polyethylene.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, medical practitioner or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.