20203 Contender - ELBOW BRACE

Contender - ELBOW BRACE

Contender Elbow is used in postoperative surgery or post traumatic stabilization of arm and elbow to provide controlled range of motion or immobilization. Telescopic mechanism for easy adjustment of the length. Extension and flexion can be adjusted and locked between -10° and 90° , with adjustments made in 10° increments. Controlled range of movement from -10° to 110° .

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Corflex, Inc
669 East Industrial Park Drive
Manchester, NH 03109,
www.corflex.com

This packaging can be recycled as paper and the bag as plastic.

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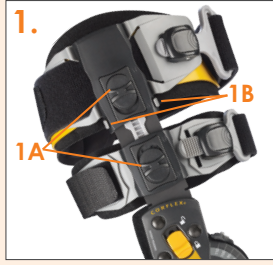
EN For best fit, modify the brace prior to application. Follow the five steps in order. Note: Initial brace setting is full elbow extension with drop lock engaged. Do not disengage drop lock or change ROM settings unless specified by the physician.

1E
Do NOT apply if safety notches are visible!

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EN Brace features a unique removable Biceps Cuff to help prevent incision site irritation or to accommodate short humeral lengths. Removal of this cuff should be only at the direction of the treating physician.

1A. Unlock both proximal lock buttons.
1B. Depress the two grey Cuff Release Buttons located on either side of the rail at the base of the Upper Cuff. Slide cuff to remove.
1C. Slide the narrow Biceps Cuff off the rail.
1D. Reapply Upper Cuff to rail. Slide Upper Cuff onto rail until the (ZipTrak) button can be engaged.
1E. If Safety Notches are visible in the upper rail the Upper Cuff is not fully engaged and brace should not be applied to the patient. Unlock the (ZipTrak) button and slide cuff further onto track until notches are no longer visible.

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EN Ensure that brace length, and cuff position have been properly adjusted before proceeding.

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
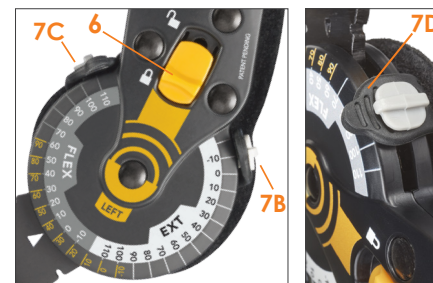
EN **6A.** Drop Lock comes set at 0° and can be locked in 10° increments from -10° to 90°.

6B. To unlock, slide gold Drop Lock Button away from hinge center. To lock, rotate hinge to one of the eleven drop lock positions and slide the gold button down toward the hinge center. Slide button can also be left unlocked for free range of motion.

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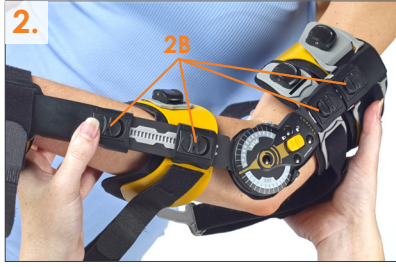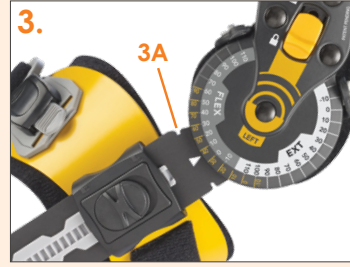
EN 2A. Position the hinge over the lateral humeral epicondyle.
2B. Unlock all four lock buttons located along the rail.
2C. Extend upper and lower rails until they match the length of the upper arm and forearm. Glide intermediate cuffs to desired location taking care to avoid surgical sites.
2D. Engage all four lock buttons.

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EN 3A. Brace uprights can be contoured to help prevent migration. Place hinge on a solid surface with bending notches above the edge to protect hinge function. Uprights can also be contoured anywhere along the exposed metal track.
3B. Apply gentle constant pressure in desired direction.

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EN 4A. Contour the Upper Cuff to the upper arm. Front section is shorter to avoid irritation in the axilla. Contour the extended rear section to the triceps.
4B. Wings of the forearm can be shaped independently for maximum contact.
4C. Adjust the Wrist Cuff to match the desired hand position for neutral, supination or pronation.

5. Adjusting Straps and Securing the Brace

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EN 5A. Shorten the straps by removing hook tip, trimming strap to desired length and reapplying alligator-style hook tip to end of strap.
5B. Insert the gray part into the quick-release lock to secure.

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EN 7A. Disengage the yellow drop lock.

7B. To set extension, remove black security band from the white button. Depress white button and slide to desired setting.

7C. To set flexion, remove black security band from the grey button. Depress grey button and slide to desired setting.

7D. Apply black security band to each button after adjustment to prevent patient tampering.

7E. Zip ties are also included for a more permanent lock. Insert zip tie through each button, tighten and trim excess.

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EN The prescription, recommendation of use and fitting of this product should only be made by a healthcare professional or orthopaedic technician.

Use the knee brace according to your fitter's instructions: Always read carefully all the instructions before using this product. It is important to follow the doctor's/fitter's instructions during the entire rehabilitation period, especially with regards to the flexion/extension stop settings. Never change the settings without the advice of your doctor/fitter.

Maintenance: Regularly check that the Velcro straps are free from dirt and that their fastening capabilities have not deteriorated. Avoid using skin cream to prevent a negative impact on the bandage. Do not use the knee brace in the bath or shower.

Special attention should be exercised if the knee brace is used by children: In such a case, both the product and its adjustments should be checked regularly by an adult. The same applies if the product is used by a person who cannot monitor the product and adjustments himself/herself due to a physical or mental disability.

Always contact your doctor/fitter: In case of doubt, if you have questions, if you are taking medication, or if you experience side effects such as skin irritation. Serious incidents shall be reported to NordiCare and the competent authority.

Indications

Indicated for postoperative or post-traumatic stabilization. Muscle/ligament injuries. Distal humerus fracture. Elbow surgery. Stable elbow fracture. **Contraindication** - Instable fractures and fractures in humerus, radius or ulna.

Materials

Brace: Lightweight Aluminum.
Cuff: Aluminum and nylon.
Padding: Polyethylene and nylon.

Washing Instructions

Wipe the aluminum brace with a soft wet cloth. Wash the padding by hand in cold water. Dry flat. Do not use bleach.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.

Size

Universal size. Available for right or left arm.

Extra

Included in the package:

- Arm sling.

To be ordered separately:

- Hand piece 20203-X5-50
- Padding 20203-X0-55
- Strap Kit 20203-30-00

Arm sling

Hand piece 20203-15-50

Product	Side	Art.no	Length orthosis (cm)	Fits upper arm with circumference	Fits forearm with circumference
Contender Elbow Brace	Left	20203-10-50	35-50	< 56 cm	< 46 cm
Contender Elbow Brace	Right	20203-20-50	35-50	< 56 cm	< 46 cm
Hand piece	Left	20203-15-50			
Hand piece	Right	20203-25-50			
Padding	Left	20203-10-55			
Padding	Right	20203-20-55			
Strap Kit	Universal	20203-30-00			