## INSTRUCTIONS FOR USE

# HumerusComfort Collum- SHOULDER IMMOBILIZER

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**EN** Machine wash at 40°C. Secure or remove all hook and loop closures before washing. Use a washing bag. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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**EN** Prescription and fitting of this product should be done by/or under the supervision of a healthcare professional. For best results, follow the instructions. Check that the Velcro straps are clean and attaches properly. Consult a healthcare professional when used in combination with other medicines or medical devices.

### Material Composition

PU Foam 45%, Polyamide 30%, Polyester 15%, Cotton 10%. Latex free.

### Size:

Universal size. The product is delivered mounted for either right or left arm.

Size	Art.no.
Universal left	20225-10-50
Universal righth	20225-20-50

Please consult our product catalogue for more information.  
For further questions, contact your doctor, health practitioner,  
medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.

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 This product meets the requirements of Medical device class 1, non sterile.

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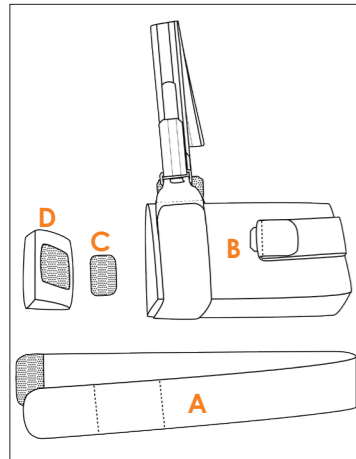
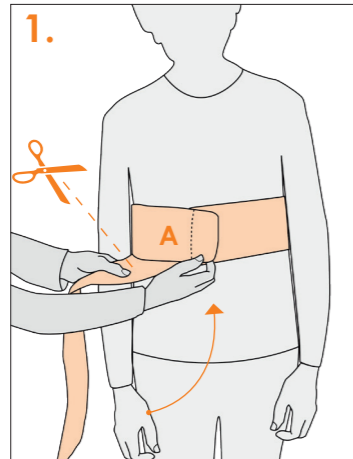
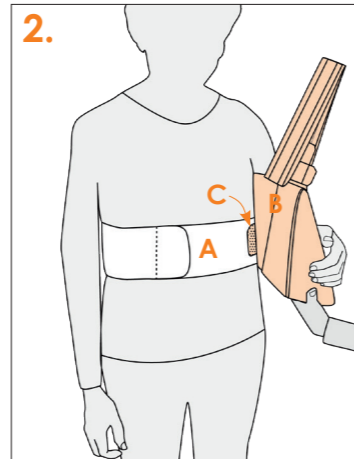
EN

**EN** Attach the chest strap **A**. Make sure the closure is easily accessible for the healthy arm. Cut the strap to the desired length.

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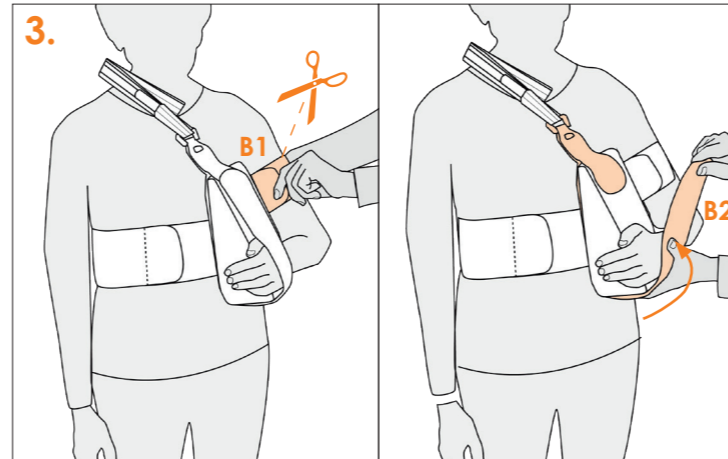
EN

**EN** Position the pillow **B** against the chest strap **A** using the Velcro fastener **C**. Make sure the arm rests on the pillow.

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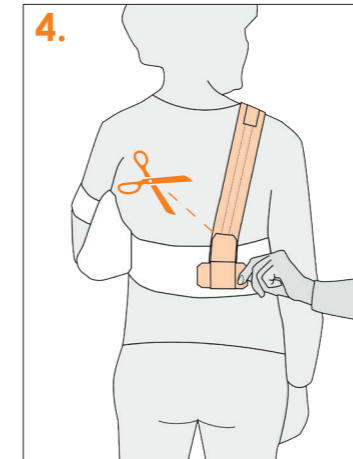
EN

**EN** Attach the arm to the upper arm strap and wristband on pillow **B**. Start by opening the upper arm strap **B1** and fasten around the upper arm. The upper arm strap can be cut if the length needs to be adjusted. Then open the wristband **B2** and secure around the hand.

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
EN

**EN** Attach the shoulder strap at the back, close to the healthy arm to reduce the pressure against the neck. The shoulder strap can be cut if the length needs to be adjusted.

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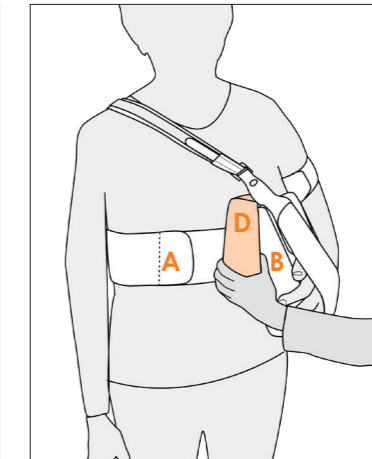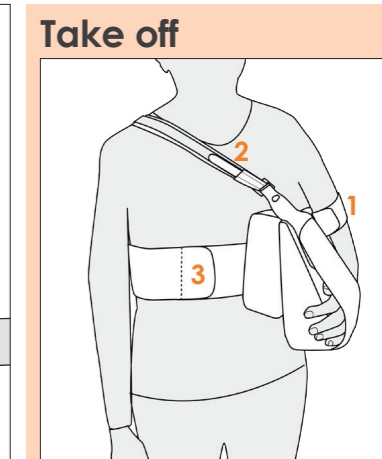
EN

**EN** If necessary the positioning pillow **D** can be placed between the chest strap **A** and the pillow **B** to achieve the desired position.

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EN

**EN** Open the upper arm strap **1**, the shoulder strap **2** and the chest strap **3** and the orthosis can be taken off in one piece.

## Put on