# Ballerina

art no 50103

POSTURE VEST

Phone +46 42 35 22 20,  
Fax +46 42 35 22 21  
info@nordicare.se,  
www.nordicare.se

Solrosvägen 1,  
SE-263 62 Viken  
Sweden

NordiCare®

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**EN** Functional posture  
vest that improves posture  
and clavicle fractures.  
Used to assist with cases  
of osteoporosis, increased  
thoracic kyphosis,  
compressed vertebrae,  
and clavicle fractures.

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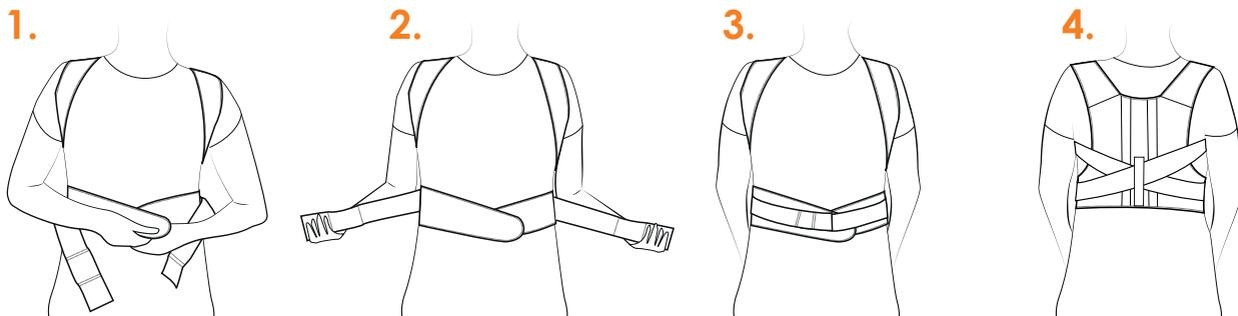
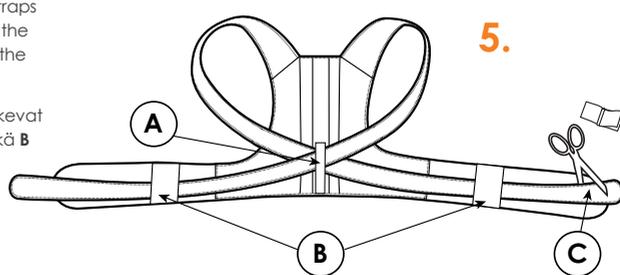
**EN** Make sure that the shoulder straps are crossed through the loop **A** in the back and through the loops **B** on the waistband.

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**EN** 1. Put on the posture bandage over the shoulders like a waistcoat.  
2. Tighten the waistband and secure the Velcro (figure 1). To avoid pressure on the diaphragm, fasten them obliquely downwards.  
3. Take hold of the adjustable straps and pull them out and forwards until the shoulders are pulled back to the appropriate position, (figure 2). Secure the Velcro fastening at the front, (figure 3).  
4. If the adjustable straps are too long, open the 'crocodile clip', and cut the strap to the required length, (figure 5 (C)).

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**EN** 1. Put on the posture vest over the shoulders like a waistcoat.  
2. Tighten the waistband so that it feels comfortable, (figure 1). Secure the Velcro fastenings.  
3. Take hold of the adjustable straps and pull them out and forwards until the shoulders are pulled back to the desired position, (figure 2). Secure the Velcro fastening at the front, (figure 3).

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### Measurement

Under bust in cm.

Size	Measurement in cm
Small	65-80
Medium	80-95
Large	95-110
X-Large	110-125

### Material

Ventilated, breathable textile material. Polyester 40%, Polyamide 37%, Polyurethane 17%, Elastane 5%, Elastomer 1%. Latex free.

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**EN** Used for advanced thoracic kyphosis. Also suitable for collar bone fractures, spinal stenosis, to achieve optimal breathing function, and in the case of compressed vertebrae.

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**EN** Not intended for people with weight >110 kg.

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**EN** If you experience pinching, chafing, or color changes in the skin, remove the brace and contact the product fitter. Product fitting, prescribing and recommendations of use should be done by or under the supervision of a qualified medical practitioner. For best results, follow the instructions. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Machine wash, 40°C (104°F). Close the Velcro. Use a washing net. Flat dry. Do not bleach, not dry clean. Do not iron or tumble dry.

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Please consult our product catalogue for more information.  
For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.