# NordiCare®

## Atle Mage

art no 50105

SPINAL ORTHOSIS

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**EN** 1. Bend the splints to spine shape and put in pockets.

2. Loosen Velcro straps. First attach the velcro straps on one side of the front part. Place the orthosis in place and fasten the velcro strap on the other side of the front part. Adjust the Velcro to the front end until the desired shape of the stomach is reached and feels comfortable.

3. Pull the straps firmly outward-forward.

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**EN** To be used during lumbago, sciatica and other usually occurred back problems.

### Heights and Measurement

To determine the size, measure hip circumference in cm.

Model	Art.no	Height front	Height back
Atle Mage, beige	50105-08	25 cm	26,5 cm

Size	Hip Circumference
Small	80 - 95 cm
Medium	95 - 105 cm
Large	105 - 115 cm
X Large	115 - 125 cm
2X Large	125 - 135 cm

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

### Material Composition

Polyamide 54%, Polyester 39%, Elastane 6%, Rubber 1%.

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**EN** Machine wash at 40°C (104°F). Remove the splints. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.