LyftPlus Special

art no 50208

ABDOMINAL BINDER

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EN Aemi-finished product to be used for manufacturing of an extra supportive elastic abdominal binder. The front is constructed to provide an efficient support and "lift" of the abdomen.

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info@nordicare.se,
www.nordicare.se

MD Class I **CE**

 This packaging can be recycled as paper and the bag as plastic.

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EN Semi-finished product - Customize and sew the product clearly.

1. Position the binder and secure the Velcro fastenings at the sides. If you have a large or hanging abdomen/hernia, it may be preferable to put the binder on while lying down. This will help the abdomen/hernia to settle in the right place and ensure that the binder provides optimal support.
2. Adjust until comfortable. Make sure it is not too tight.

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- EN** To be used post-operatively after abdominal surgery, to provide relief for surgical wounds and to prevent incisional hernia, also for pain relief and early mobilization of patients. Can also be used as an ostomy or hernia support.

Counter indication: incarcerated hernia.

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Heights and Measurement

LyftPlus Special is 36 cm high at front and 25 cm high at back. For correct size measure the hip circumference in cm.

Size	Hip Circumference
M	90 - 105 cm
L	100 - 115 cm
XL	110 - 125 cm
2XL	120 - 135 cm

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- EN** Prescribing, product fitting and recommendations of use should be done by or under the supervision of a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Serious incidents shall be reported to NordiCare and the competent authority.

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- EN** Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dryclean. Do not iron. Do not tumble dry.

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Accessory

To be ordered separately, if necessary for individual adaption.

Product	Art.no:
Elastic edge binding 22 mm, Beige, Approx. 150 m	91121-08-00
Folded velour ribbon 21 mm, Beige, 25 m	91122-08-21
Velour Active, Beige, 1,3 x 1 m	91130-08-00