INSTRUCTIONS FOR USE

StomiMini – STOMA BINDER

art no 50213

Application

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EN 1. Apply the fixation band around the intended placement.

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EN Circumference of intended placement. Universal size, fits circumference measuring up to 120 cm.

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EN Machine wash at 60°C. Secure the hook closures before washing. Use a washing bag. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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Material

Polyamide 55%, Polyester 35%, Elastane 10%. Latex free.

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EN To be used as a stoma band for fixation of ostomy pouches. Suitable to use for ileo-, colo- or urostomy. Not suitable as support for parastomal hernias.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Check that the Velcro is are clean and attaches properly. Consult medically trained personnel if using this product during medication and/or in combination with aids of assistance. Serious incidents shall be reported to NordiCare and the competent authority.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner or medical equivalent - or Nordicare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.