Compression Garments by NordiCare

Daisy

art no 51204


BODY


Compression class 1 (15-21 mmHg)

Available colors: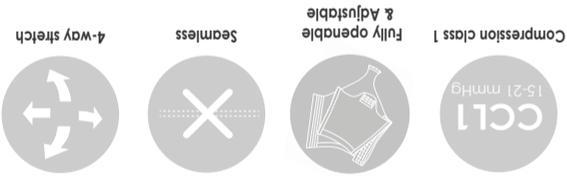

Size	Waist cm	Hip cm
XS	62-68	86-91
S	69-75	92-97
M	76-83	98-103
L	84-92	104-111
XL	93-101	112-119
2XL	102-110	120-127

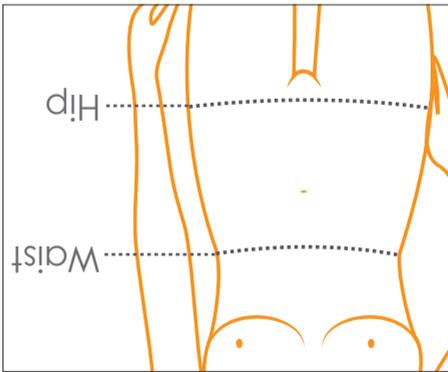NordiCare®
 Solrosvägen 1,
 SE-263 62 Viken
 Sweden
 Phone +46 42 35 22 20,
 Fax +46 42 35 22 21
 info@nordicare.se,
 www.nordicare.se

OEKO-TEX®
 STANDARD 100
 CONFIDENCE IN TEXTILES
 6234-7001 DT1
 Tested for harmful substances.
 www.oeko-tex.com/bunddt100

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EN Compression body in knitted seamless material, double material over the abdomen/pocket. Recommended after surgery and/or liposuction on the abdomen. Fully openable/easy to apply immediately after surgery with hook and eye closure on both sides and also at the crotch. Silicone waistband.

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EN Recommended after plastic surgery. Compression improves circulation and reduces the risk of post-operative swelling, leading to better healing and even pain relief. Corresponding compression class 1.

Contraindications: Infection. Arterial circulatory disorders. High risk of thrombosis.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Serious incidents shall be reported to NordiCare and the competent authority.

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EN For best fit and correct compression it is important that measurements are taken carefully and size chosen according to the measuring table. The fit of the garment should be tight but comfortable. If the patients measurements differ from the measurement table, the garment should not be used. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required.

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EN Machine wash at 40°C (104 F). Can be washed at a higher temperature if necessary, max. 60°C (140 F). The product will last longer if washed at a lower temperature. Use a laundry bag. Do not tumble dry. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.