ImmoKid

art no 70510

KNEE IMMOBILIZER

Measurement

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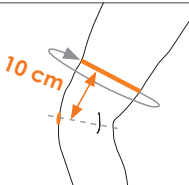
EN Circumference of thigh, measure 10 cm/4 inch above knee, from center/ mid-patella.

	Size	Length cm	Circumference cm (fits up to)
Child	2XS	15	28
	XS	23	35,5
	S	27	35,5

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
EN Circumference of thigh, measure 10 cm/4 in above knee, from center/ mid-patella.

	Size	Length cm	Circumference cm (fits up to)
Child	2XS	15	28
	XS	23	35,5
	S	27	35,5

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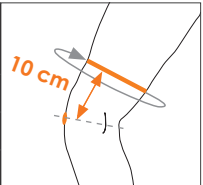- EN** 1. Open knee immobilizer and remove both straight aluminium stays.
2. Position knee immobilizer with the widest end around thigh and the contoured stays centered in back of leg.
3. Re-attach both straight aluminium stays on each side of leg.
4. Attach the elastic straps, starting at bottom and moving up. Attach straps where comfortable, making sure not to inhibit circulation.

Material Composition

50% Neoprene, 25% Nylon, 10% EVA Laminate, 5% Delrin, 5% Elastic, 5% Stretch UBL. Aluminum stays. Latex free.

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EN For immobilization of the knee joint after operation or trauma.

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EN Hand wash max 30°C. Use washing bag. Dry flat. Secure all hook and loop closures before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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