# INFORMATION FOR Stratus - AIRWALKER

art no 80225

## Recommendation

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**EN** These products are only to be used/applied by medical specialists. In order to dispose of the product, comply with the legal norms of your country. If any serious incident that has occurred in relation to the device should be reported to NordiCare and the competent authority of the Member State in which the user and/or patient is established.

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**EN** Please consult with your physician if any allergic symptom occurs. Do not use more than 24 hours of continuous.

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**EN** Class I medical device, non sterile, no measurement function. This product fulfils the Medical Device Regulation (EU) 2017/745.

## Material Composition

45% Polypropylene (PP),  
15% Termoplastic Rubber (TPR),  
5% Termoplastic Polyurethane (TPU),  
5% EVA, 10% Polyester, 5% Nylon,  
15% Polyurethane (PU).  
Latex-free.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se).  
For further questions, contact your doctor, health practitioner or medical equivalent - or  
NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se)

### Wellcare

Zhangpu Ortholife Rehabilitation Co., Ltd.  
No.6 West Lanli Rd., Suian Development  
District, Zhangpu County, Fujian Prov.,  
363200, China

### SUNGO Europe B.V.

Fascinat Boulevarde 522, Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands

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Solrosvägen 1, SE-263 62 Viken, Sweden.  
Tel +46 42 35 22 20, Fax +46 42 35 22 21  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)

**MD** Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

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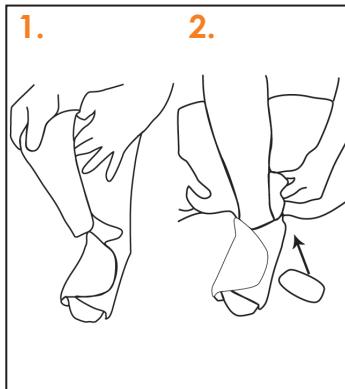
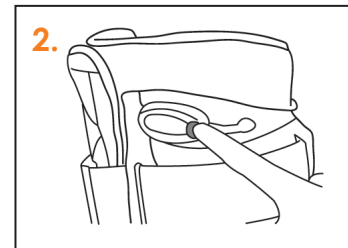
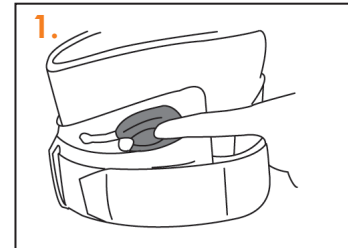
- EN** 1. Place foot and leg inside foam liner and fasten it snugly.  
2. You may put pad kits to supplement inner space or between walker frame and liner.  
3. Place wrapped foot into Walker. Place front panel to ensure protection and support.  
4. Fasten ankle, foot and calf strap in order. Tighten straps until snug and comfortable.

How To Adjust Aircell Pressure

1. Press the air pump to get inflation control.
2. Press the valve for deflation.

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**EN** Stable foot or ankle fracture. Severe ankle sprain. Post surgery application.  
**Contraindications:** Paraesthesia. Severe varicose vein. Skin disorders or injuries in the relevant area. Impaired lymphatic drainage.

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**EN** Provide support and comfort following trauma or post-operative procedure.

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**EN** Close all straps before washing. Hand wash in cold water with mild soap. Dry clean, bleach or tumble is prohibited. Natural dry, and do not expose this brace to any heat source.

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**EN** A heel wedge can be used as an accessory if an increased height is wanted for the plantarflexion. Independent wedges are assembled together to the desired height. Toe-, hygiene- and rainprotection is also possible to order.